

GENDRON

Bariatric Rehab Leadership





Total Solutions for Bariatric Patient Care®

Founded in 1872, Gendron, Inc., is the oldest manufacturer of assistive medical devices in the United States. For more than a decade, Gendron has focused our innovative engineering, quality manufacturing and customer service on providing equipment specifically tailored to the rigorous requirements of the bariatric patient. Today Gendron is the U.S. leader in providing total solutions for bariatric patient care.

Our leadership begins with extensive research, design and testing. We understand the needs of patients, as well as their caregivers, and providers. Each bariatric patient, who could weigh from 350 to more than 1000 pounds, creates extraordinary demands on equipment strength, durability, design and materials. Responding to the needs of our customers, we develop specialized equipment to help manage bariatric patient care in all environments – acute care, long-term care, and home care.

Our total bariatric solutions approach is the key to patient comfort and safety during every phase of care. Based upon years of experience, we have learned how each type of care environment requires particular products. At home, for example, your client may require bariatric bathing, toileting and ambulatory products in addition to a home care bed and wheelchair. For acute care and long-term care settings, we provide complete bariatric patient suites. We also design a complete line of equipment for the caregiver as well as the patient, to ease difficult patient transfer and transport. These bariatric products include trauma/ER stretchers, transport stretchers, transport/procedures chairs, stretcher beds and the Extra Care™ bed with sit-to-stand and cardio chair positioning.

Finally, our leadership embraces responsive customer service. We help caregivers and providers determine the optimum equipment configuration and accessories for each patient. You will appreciate our quick turnaround. We routinely equip a complete bariatric patient room in days. Based on more than 130 years of experience, you can count on Gendron working closely with you to provide individualized solutions for you and for your bariatric patients.

BARIATRIC PRODUCTS CATALOG



■ BARIATRIC BEDS & ACCESSORIES

Maxi Rest™ Bariatric Bed	750-1000 lb	4-5
Extra Care™ Bariatric Bed	1000 lb	6
Home Care Bed	650 lb	7
Patient Room Recliner	850 lb	8
Bariatric Stretcher Bed	1000 lb	9
Bariatric Support Surfaces	1000 lb	10
Maxi Lift™ Bariatric Trapeze.....	850 lb	11
Extra Care™ Bariatric Lift.....	700 & 1000 lb	11

■ BARIATRIC MANUAL & POWER WHEELCHAIRS & ACCESSORIES

Measuring & Ordering Guidelines.....		12-13
Tracker II™ & Regency XL™	350 & 450 lb	14
Tracker II™ & Regency XL™ Recliner	350 lb	14
Regency XL 2002™	600 lb	15
Regency XL 6500Q™	600 lb	15
Regency XL 2000™ & Recliner	700, 750 & 850 lb	16
Regency DX™ & LX™ Folding Frame Power Wheelchairs	450 & 600 lb	17
Regency XLC™ Power & Tilt In Space--CUSTOM BUILT	700-1100+ lb	18
Wheelchair Accessories		19-23
Profiler XL™ Bariatric Pressure Relief Seating Support Cushion		23

■ BARIATRIC BATHING, TOILETING & AMBULATORY

Wheeled Shower Commode Chair	750 lb	24
Bed Side Commode	700-1000 lb	24
Commode	850 lb	24
Bath Bench	500 lb	25
Transfer Bench.....	650 lb	25
Transfer Bench.....	500 lb	25
Bariatric Walkers.....	750 & 850 lb	26
Bariatric Seating - Chair With Hardwood Frame	750 lb	26

■ BARIATRIC LONG-TERM & ACUTE CARE EQUIPMENT

Extra Care™ Transport Chair.....	850 lb	27
Extra Care™ Trauma/ER Stretcher.....	1000 lb	28
Bariatric Transport Stretcher	1000 lb	28

■ MRI SUITE TRANSPORT PRODUCTS

MRI Transport Wheelchair	250-350 lb	29
MRI Transport Stretcher.....	350 lb	29

■ PRONE STRETCHER

BARIATRIC BEDS: 750-1000 LB. WEIGHT CAPACITY



Maxi Rest™ Bariatric Bed

(Shown with optional over-bed trapeze)

- **Weight Limit: 750-1000 pounds**
- Deck Size:
 - Widths - 48" or expandable 39", 48" or 39", 48", 54"
 - Length - 84"
- Deck Height: Adjustable 15" - 29"
- Fully electric with battery back-up
- 3-Function or 5-Function
- Split frame or transport frame
- Over-bed trapeze optional
- Weighing scales optional

WARRANTY:

Five-year warranty on the bed frame & deck.
One-year warranty on all electrical components & controls.

BARIATRIC BEDS: 750-1000 LB. WEIGHT CAPACITY

Maxi Rest™ Bariatric Bed (Shown with optional weighing scale)



Maxi Rest™ Bariatric Bed - Features



Leg Lift



Width Extension Panels



Length Extension



Cord Wrap

BARIATRIC BEDS: 1000 LB. WEIGHT CAPACITY



Extra Care™ Bariatric Bed

- **Weight Limit: 1000 pounds**
- Deck Size:
Widths - Adjustable 39", 48", 54"
Length - 84"
- Deck Height: Adjustable 19.5"-39"
- Sit to stand and cardio chair positioning
- Foot-end egress
- Weighing scales included
- 8" all-surface casters, for ease of transporting
- Four-wheel central lock and steer
- Two-position swing-down side rails



WARRANTY:

Five-year warranty on the bed frame & deck.
One-year warranty on all electrical components & controls.

BARIATRIC BEDS: 650 LB. WEIGHT CAPACITY



Home Care Bed

- **Weight Limit: 650 pounds**
- Deck Size:
Widths - 39" or 48"
Length - 84"
- Deck Height: Adjustable 15.75" - 32.5"
- 3-Function, fully electric operation
- Optional portable battery pack for emergency operation
- Functions: High/low, fowler back rest, leg lift
- Headboard, footboard and side rails are standard
- Split frame construction for ease of delivery and set-up
- Quick connect cables for easy assembly
- Frame connector pins tethered to frame to prevent loss



Battery Pack

WARRANTY:

Five-year warranty on
the bed frame & deck.
One-year warranty on
all electrical
components & controls.

BARIATRIC RECLINER: 850 LB. WEIGHT CAPACITY



Patient Room Recliner Model Series 7150

- **Weight Limit:**
850 pounds
- Seat Width: 29.25"
- Seat Depth: 19"
- Seat to Floor Height: 20.5"
- Back Style: 32" high, reclines to 40°
- Locking caster wheels
- Attendant-operated recline
- High-density cushioned seat and back for user comfort
- Leg rest articulates as back reclines
- Colors: Navy, burgundy, grey, brown



WARRANTY:

Five-year warranty on the chair frame.

BARIATRIC STRETCHER BED: 1000 LB. WEIGHT CAPACITY



Bariatric Stretcher Bed **Model 5154DX**

- **Weight Limit: 1000 pounds**
- Deck Size:
 - Widths - 39", 48", 54"
 - Lengths - 80", 84"
- Deck Height: Adjustable 16"-29.5"
- 5-Function, fully electric
- Battery back-up
- Optional weighing scales (shown)
- 8" diameter all-surface casters, for ease of transporting
- Swing-down side rails
- Head/foot boards, side rails are standard
- Four-wheel central lock and steer

WARRANTY:

Five-year warranty on the bed frame & deck.
One-year warranty on all electrical components & controls.



Bariatric Support Surfaces

- **Weight Limit: 1000 pounds**
- Choose from 39", 48" or 54" wide surfaces
- Low air loss
- Alternating pressure
- Combination low air loss/alternating pressure

ONE-YEAR
WARRANTY

BARIATRIC TRAPEZE/BARIATRIC LIFT



Maxi Lift™ Bariatric Trapeze Model 5160

- **Weight Limit:**
850 pounds
- Overall Height: 68"
- Portable over-bed trapeze with adjustable trapeze bar
- Low profile base fits under most beds
- Floor anchors recommended

WARRANTY:

Three-year warranty
on the frame.
One-year warranty on
all other parts.



Extra Care™ Bariatric Lift Model GT700


- **Weight Limit: 700 pounds**
- Lift Range: 31"-76"

Model GT1000 (not shown)

- **Weight Limit: 1000 pounds**
- Lift Range: 39"-76"
- Fully electric lift operation with pendant and push-button controls
- Manually adjustable retractable base legs for safety and stability during transfer
- Size 3XL sling included
- Weighing scales available

**THREE-YEAR
WARRANTY**

WHEELCHAIR MEASURING GUIDELINES

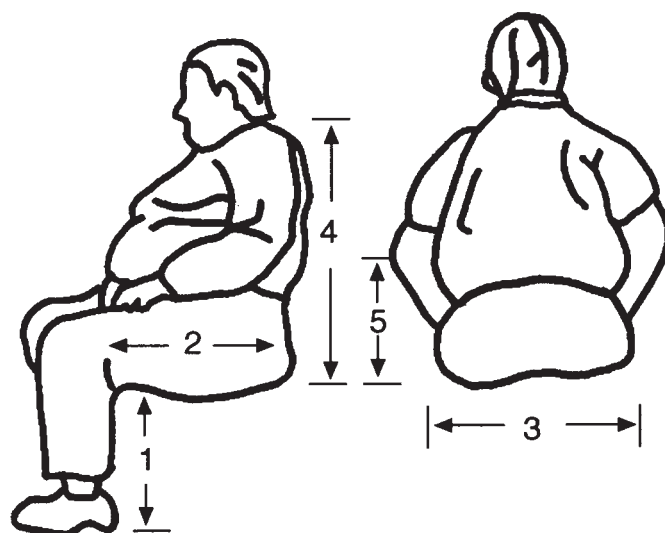

Measuring Proper Fit & Wheelchair Construction

1. Seat distance from floor (Seat Height) _____
2. Depth of seat from front-to-rear (Seat Depth) _____
3. Overall width of seat (Seat Width) _____
4. Seat to top of back (Back rest Height) _____
5. Seat to top of armrest (Armrest Height) _____

Back Angle (degrees) _____
If the seat plane is not parallel to the floor,
indicate the distance from the front and rear
of seat to floor.

- Front Edge to Floor _____
- Rear Edge to Floor _____

Please check for environmental access restrictions.
Tell us the maximum overall width, wheel to wheel, to ensure
passage through doorways, ramps, etc.

**Gendron recommends that specifications to build each wheelchair be
supplied by a physician, physical therapist or other qualified medical professional.**

WHEELCHAIR MEASURING GUIDELINES

Actual Body Weight.

If client body type is pear-like, consider potential of stable weight and stable wheelchair width over time. If client body type is apple-like, consider potential for fluctuating weight. If deciding between two sizes, consider opting for larger size and potentially wider wheelchair.

Custom Measuring for the Wheelchair User.

If possible, have the client sit on a hard surface such as a therapy mat or a square of firm plywood. The client's thighs should be level and not upward or downward sloping relative to the hip joint. The lower leg should be in a natural, vertically oriented posture. This posture allows for easy access to measure the client from a true postural set, otherwise not possible from a soft surface. Once the overall height is determined, the next consideration in wheelchair measurement is the pelvis, as this is the primary weight-bearing surface.

1. Seat Height: With feet flat on the floor and the shin in a vertical posture, measure from the back of the heel to the underside of the knee. The client should wear their typical footwear, and again the thighs should be level relative to the hip joint prior to measurement. This will allow for proper foot rest length and overall wheelchair height which is so vital in wheelchair propulsion, as the obese population tend to achieve propulsion by a combination of hand and foot use.

Wheelchair floor to seat height is also critical for sit to stand activities. For individuals who are primarily exercise ambulators, a lower seat height may be indicated allowing community propulsion, while individuals who ambulate to vital rooms or bathrooms, a higher seat height may be indicated. Recall that wheelchair cushions will add to the height of the finished sitting surface.

2. Seat Depth: Measure from the back of the buttocks to within 2 inches of the back of the knee. The completed wheelchair should allow for approximately 2 inches of space between the back of the client's knee and the front of the wheelchair seat thereby preserving circulation to the lower leg while maximizing the client's weight-bearing surface and leg mobility during foot-assisted propulsion. The seat surface should support the entire gluteal region.

3. Seat Width: Measure widest part of the client in seated posture. Again consider apple versus pear. A pear-shaped individual having greater gluteal femoral weight distribution may be widest near the front edge of the seat. Excessive wheelchair width will restrict mobility about environmental barriers, increase difficulty of turning and decrease arm support with resulting potential for back pain. The completed wheelchair will allow for approximately 1 to 2 inches of width on either side of the client for winter clothing, client weight shifting during pressure relief and, if necessary, room for possible lift devices such as slings. On occasion, clients may opt to remove wheelchair push rims to accommodate narrow barriers.

4. Back Rest Height: Measure from the seat surface to mid-shoulder-blade height. The back rest generally should reach to mid-shoulder-blade level in height and support the apex of the client's back to diminish potential for postural back pain and provide for adequate pressure relief while allowing maximal shoulder-blade mobility. If the client is in a reclining chair then additional upper thoracic support may be indicated. More agile clients may prefer a back rest that is positioned vertically just 1 inch below the shoulder blade, allowing for maximal upper body mobility over their lower trunk in sitting postures. If a client may have localized excessive tissue bulk causing partial contact to their back rest, a strap or laced back rest may be necessary to provide sufficient support for that unique body type.

5. Armrest Height: Measure directly from the sitting surface to the bent elbow having the forearm parallel to the seat. Recall that a seat cushion may add to the height of the seat and equally add to the height of the armrest. Appropriate armrest height is determined from this measurement and is important for decreasing neck and thoracic back pain by providing adequate support for the shoulder girdle. Remember that respiratory-impaired individuals derive increased respiratory support by leaning upon their forearms and thereby increase depth of breathing by reverse action of upperbody muscles. This is common in the obese client with respiratory compromise or congestive heart failure.

** Measuring Guidelines reprinted with permission of Michael Dionne, PT, Choice Physical Therapy, Inc.*

MANUAL WHEELCHAIRS: 350 & 450 LB. WEIGHT CAPACITY



Tracker II™ & Regency XL™

Tracker II™ Basic 5300 Series
Tracker II™ Deluxe 6300 Series
• **Weight Limit: 350 pounds**

Regency XL™ 5400 Series
Regency XL™ 6400 Series
• **Weight Limit: 450 pounds**

- Seat Widths: 20", 22", 24"
- Seat Depth: 18"
- Seat to Floor Height:
Basic - 19.5"
Deluxe – Adjustable 19.5", 18", 16"
- Back Style:
Basic - Permanent 16" above seat level
Deluxe – Adjustable 16", 17", 18"
- Arm Style: Removable desk or full length

Tracker II™ & Regency XL™ Recliner

Tracker II™ Deluxe Recliner 6300 Series
• **Weight Limit: 350 pounds**

Regency XL™ Deluxe Recliner 6400 Series
• **Weight Limit: 450 pounds**

- Seat Widths: 20", 22", 24"
- Seat Depth: 18"
- Seat to Floor Heights: 16", 18"
- Back Style: Power-assist reclining back, 36" high
- Arm Style: Removable desk or full length



WARRANTY:

Limited lifetime
warranty on side
frames & cross braces.
One-year warranty on
all other components.

MANUAL WHEELCHAIRS: 600 LB. WEIGHT CAPACITY



Regency XL 2002™

Bariatric Manual Wheelchairs - Model Series 5624, 5626, 5628

- **Weight Limit: 600 pounds**
- Seat Widths: 24", 26", 28"
- Seat Depth: 18"
- Seat to Floor Height: 19.5"
- Back Style: Fixed height 16" with 5° back angle
- Arm Style: Removable desk or full length

Regency XL 6500Q™

Bariatric Manual Wheelchairs

- **Weight Limit: 600 pounds**
- Seat Widths: 20", 22", 24"
- Seat Depth: 16"
- Seat to Floor Height: 20.5"
- Back Style: 15.25" high
- Arm Style: Full length permanent



WARRANTY:

Limited lifetime warranty on side frames & cross braces. One-year warranty on all other components.

MANUAL WHEELCHAIRS: 700, 750 or 850 LB. WEIGHT CAPACITY



Regency XL 2000™

Series 6700

- **Weight Limit: 700 or 750 pounds**
- Seat Widths:
20", 22", 24", 26", 28", 30", 32", 34"
- Seat Depths: 18", 20", 22"

Series 6800

- **Weight Limit: 850 pounds**
- Seat Widths: 26", 28", 30", 32", 34"
- Seat Depths: 20", 22"
- Seat to Floor Heights: 15", 16", 17", 18"
- Back Style:
Adjustable height 16", 17", 18"
- Dual square cross braces
for patient support
- Super hemi seat to floor height available



Regency XL 2000™ Recliner

Series 6700

- **Weight Limit: 700 or 750 pounds**
- Seat Widths:
20", 22", 24", 26", 28", 30", 32", 34"
- Seat Depths: 18", 20", 22"

Series 6800

- **Weight Limit: 850 pounds**
- Seat Widths: 26", 28", 30", 32", 34"
- Seat Depths: 20", 22"
- Seat to Floor Heights:
15", 16", 17", 18"
- Back Style:
Power-assist reclining back, 36" high
- Dual square cross braces
for patient support
- Super hemi seat to floor height available

WARRANTY:

Limited lifetime
warranty on side
frames & cross braces.
One-year warranty on
all other components.

POWER WHEELCHAIRS: 450 or 600 LB. WEIGHT CAPACITY



Regency DX™ (Folding Frame)

Series 7218

- **Weight Limit: 450 pounds**

Regency LX™ (Folding Frame)

Series 7518

- **Weight Limit: 600 pounds**

- Seat Widths: 20", 22", 24" or 24", 26", 28", 30"
- Seat Depths: 18", 20"
- Seat to Floor Height: 20.5"
- Back Style: Permanent 18" high
- Battery tray tucks under seat
- Variable speed control
- By removing battery and battery tray, wheelchair folds for transport

WARRANTY:

Limited lifetime
warranty on side
frames & cross braces.
One-year warranty on
all other components.

POWER OR MANUAL WHEELCHAIRS: 700 - 1100+ LB. WEIGHT CAPACITY

**WARRANTY:**

Limited lifetime warranty on side frames & cross braces. One-year warranty on all other components.



Regency XLC™ Power & Tilt In Space CUSTOM BUILT to Exact Client Requirements Bariatric Power or Manual Wheelchairs

- **Weight Limit: 700–1100+ pounds**
- Seat Widths: 20", 21", 22", 23", 24", 25", 26", 27", 28", 29", 30", 31", 32", 33", 34", 35", 36", 37", 38", 39", 40"
- Seat Depths: 18", 19", 20", 21", 22", 23", 24", 25", 26"
- Seat to Floor Heights: 14", 15", 16", 17", 18", 19", 20", 21"
- Back Style: Permanent or Manual Recliner
16", 17", 18", 19", 20", 21", 22", 23", 24", 25", 26", 27", 28" high
- Our custom design allows the minimum overall width possible for environmental access

BARIATRIC WHEELCHAIR ACCESSORIES



Rear Mounted Anti-Tip Brackets

Model Series 85R

Designed for greater rear end stability.



Wheelchair Safety Belt

Model Series 80


Available with auto buckle or Velcro fastener.



Quad Release Swing Away Foot Rests

Model Series 33 (with composite footplates, 33-c)

Fits all Gendron wheelchairs with the "Q" designation. Cam action quad release lever locks the foot rests into place. Releasing the lever allows the foot rests to swing away and be removed. Standard features include aluminum footplates and thru-bolt extension adjustment.

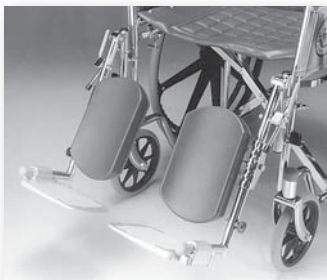

Quad Release Swing Away Elevating Leg Rests

Model Series 36 (with composite footplates, 36-c)

Fits all Gendron wheelchairs with the "Q" designation. Cam action quad release lever locks the leg rests into place. Releasing the lever allows the leg rests to swing away and be removed. Standard features include seat level elevating pivot, deep contoured cushioned vinyl calf pads, aluminum footplates and thru-bolt extension adjustment.

WARRANTY:

All Gendron accessory components are guaranteed for one full year from date of purchase.

BARIATRIC WHEELCHAIR ACCESSORIES



Oxygen Tank Holder

Model Series 12

Fits most wheelchairs.
Lock knob holds cylinder securely in place.



IV Rod/Holder

Model Series 95

Fits most wheelchairs.
Maximum IV rod height extension is 30 inches.



Oxygen Tank Holder

Model Series 13

Designed for use on reclining wheelchairs.
Holder is positioned not to interfere with the wheelchair back when reclined.



IV Rod/Holder

Model Series 14

Designed for use on reclining wheelchairs.
IV rod/holder remains in a stationary position regardless of the wheelchair back position.
Maximum IV rod height extension is 30 inches.



Combination IV Rod/Holder & Oxygen Tank Holder

Model Series 1295

Fits most wheelchairs.
Maximum IV rod height extension is 30 inches.



Heavy Duty IV Rod/Holder

Model Series 1294

Extra large diameter IV pole supports pumps and monitors.

WARRANTY:

All Gendron accessory components are guaranteed for one full year from date of purchase.

BARIATRIC WHEELCHAIR ACCESSORIES



Overhead Anti-Theft Loop At 77 inches above floor level, the overhead loop deters theft by increasing wheelchair visibility and preventing the wheelchair from folding. Available for most Regency™ wheelchair seat widths, each overhead loop includes two IV hangers. Optional oxygen tank holder is also available.

Anti-Theft Loop/IV Holder

Model Series 1296 for Seat Widths: 16" to 20"

Model Series 1297 for Seat Widths: 20" to 24"

Anti-Theft Loop/Oxygen Tank Holder


Model Series 1298 for Seat Widths: 16" to 20"

Model Series 1299 for Seat Widths: 20" to 24"



Wheel Lock Extensions

Model Series 86 - Secured with a lock knob and fits over the existing wheel lock.

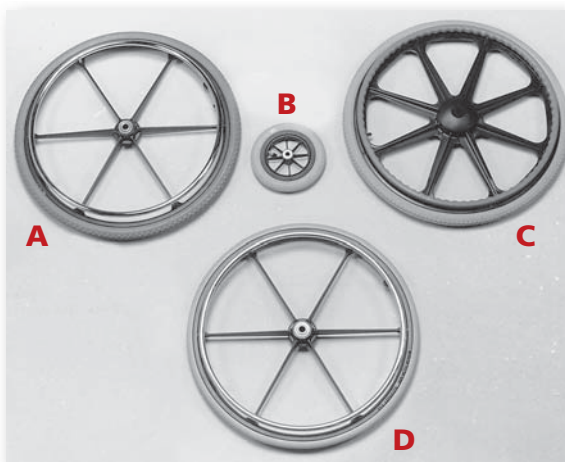

Wheels

A. Model Series 48

Cast Metal 24" Mag Wheel with pneumatic tire and chrome-plated steel handrim.

B. Model Series 51

Composite 8" Wheel with pneumatic tire.

C. Model Series 50

Composite Mag 24" Wheel pneumatic tire and molded composite handrim.


D. Model Series 45

Cast Metal 24" Mag Wheel with solid tire and black metal handrim.

WARRANTY:

All Gendron accessory components are guaranteed for one full year from date of purchase.


BARIATRIC WHEELCHAIR ACCESSORIES

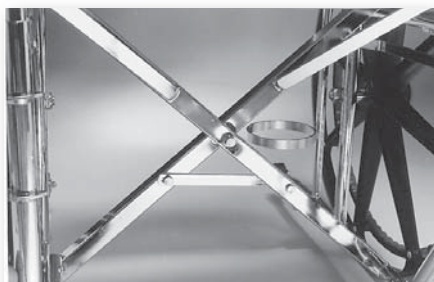

Anti-Fold Attachment

Model Series 84

Bolts to both cross braces to prevent folding of the wheelchair.

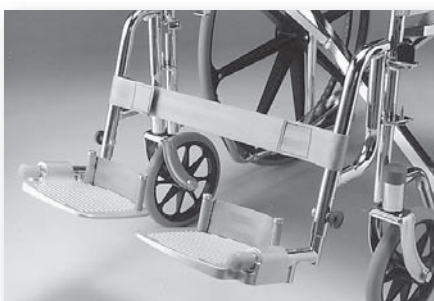

Leg Guard

Model Series 59

Constructed of 2-inch-wide webbing with Velcro fasteners.

Heel Loops

Model Series 83

To keep patient's feet positioned on the footplates. Constructed of 2-inch-wide webbing.



Extension Headrest


Model Series 72

Designed for fixed back wheelchairs, the mounting brackets attach to the pusher handles. The headrest adds 18 inches to the height of the wheelchair back.

WARRANTY:

All Gendron accessory components are guaranteed for one full year from date of purchase.

BARIATRIC WHEELCHAIR ACCESSORIES

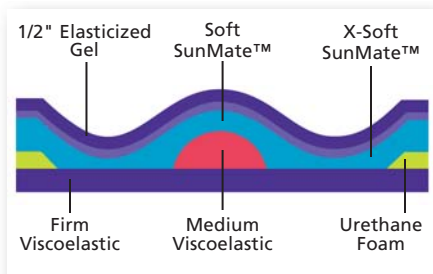

Profiler XL™

Bariatric Pressure Relief Seating Support Cushion

Combination gel/foam lamination for the bariatric client.

Available with or without hip abductor.



Solid Wheelchair Back Insert

Model Series 75

Rigid back with foam liner. Loops fit over push handles to keep the insert in place.

Solid Wheelchair Seat Insert

Model Series 94 (not shown)

Rigid base with foam liner. The insert rests on the seat tubes and provides a rigid seating surface. Insert must be removed to fold the wheelchair.



Name Stenciling

Custom name stenciling for wheelchair backs. Maximum 13 characters per line, maximum three lines.

WARRANTY:

All Gendron accessory components are guaranteed for one full year from date of purchase.

BARIATRIC COMMODES: 700-1000 LB. WEIGHT CAPACITY



Wheeled Shower Commode Chair

Model Series 5200, 5215, 5225

- **Weight Limit: 750 pounds**
- Seat Widths: 24", 26" or 28"
- Seat Depth: 20"
- Seat to Floor Height: 21"
- Arms: Removable arms
- Cushioned seat and back for added comfort and support
- Hook-on removable footrests
- Corrosion resistant
- Hygiene access
- Rear anti-tip attachments



Bed Side Commode

Model Series 5232, 5233

- **Weight Limit: 700 - 1000 pounds**
- Seat Widths: 20", 24", 26", 30", 36"
- Seat Depth: 20"
- Seat to Floor Heights: 20" or adjustable 20", 21", 22", 23", 24"
- Arms: Removable drop down arms
- Cushioned seat
- 36"-wide model rated at 1000 lbs.
- Optional wheel kit available



Commode

Model Series 5135

- **Weight Limit: 850 pounds**
- Seat Width: 26"
- Seat Depth: 19"
- Seat to Floor Height: 21"
- Arms: Dual swing away arms

WARRANTY:

Three-year warranty
on the frame.
One-year warranty
on all other parts.

BATH & TRANSFER BENCHES: 500-650 LB. WEIGHT CAPACITY



Bath Bench

Model Series 5195

Weight Limit: 500 pounds

Seat Width: 20"

Seat Depth: 11.5"

Seat to Floor Height: 14"-18"



Transfer Bench

Model Series 5145

• **Weight Limit: 650 pounds**

• Seat Width: 44"

• Seat Depth: 15.5"

• Seat to Floor Height: 18.5"



Transfer Bench

Model Series 5190

Weight Limit: 500 pounds

Seat Width: 27"

Seat Depth: 16"

WARRANTY:

Three-year warranty
on the frame.
One-year warranty on
all other parts.



WALKERS & SEATING: 750-850 LB. WEIGHT CAPACITY



Folding Bariatric Walkers

Model Series:

5176, 5176-1 (With Wheel Kit)

- **Weight Limit:**
750 pounds
- Shown with optional wheel kit (Note: Optional wheel kit reduces weight capacity to 650 lbs.)



Non-folding Bariatric Walkers

Model Series:

5175, 5175-1 (With Wheel Kit)

- **Weight Limit:**
850 pounds
- Optional wheel kit (reduces weight capacity to 500 lbs.)



Chair With Hardwood Frame

- **Weight Limit: 750 pounds**
- Seat Widths: 25", 30", 40"
- Seat Depth: 19"
- Seat to Floor Height: 18" or optional leg height adjustment 18", 19", 20", 21", 22", 23"
- Back Style: 20" from seat level
- Sit to stand position available with adjustable legs



Adjustable
Leg Height

WARRANTY:

Three-year warranty on the frame.
One-year warranty on all other parts.

Extra Care™ TRANSPORT CHAIR: 850 LB. WEIGHT CAPACITY



Extra Care™ Bariatric Transport Chair Model 6750

- **Weight Limit: 850 pounds**
- **Fully electric operation with pendant control**
- Seat Width: 29.5"
- Seat Depth: 22"
- Seat to Floor Height: 24" to top of seat cushion
- Back Style: Reclining
- **In stretcher or flat position, surface is height adjustable 24.5" to 32.5"**
- Arms: Drop down side rails
- Leg rests elevate independently
- Approximately 12 cycles between battery charges
- 8"-diameter all-surface casters, for ease of transporting
- Four-wheel central lock and brake
- Low shear fabric standard
- One-touch chair to stretcher position

WARRANTY:

Five-year warranty
on frame.
One-year warranty
on all electrical
components.

Extra Care™ STRETCHERS: 1000 LB. WEIGHT CAPACITY



Extra Care™ Bariatric Trauma/ER Stretcher Model 1600

- **Weight Limit: 1000 pounds**
- **Fully electric operation**
- Litter Size:
 - Width 37" (between side rails)
 - Overall width 39"
 - Length 80"
- Litter Height: 20" - 28"
- Fowler back rest at 55°
- Knee gatch
- Pull/push handles, self-storing
- Fold-down side rails
- Approximately 12 cycles between battery charges
- All surface 8" casters
- Four-wheel central locking
- Low shear fabric mattress cover



Bariatric Transport Stretcher Model 1190

Weight Limit: 1000 pounds

- Patient Surface: Width 35", Length 80"
- Overall width 40"
- Deck Height: 29.5"
- Back Style: Foot pedal control hydraulic fowler back rest
- 3-Position, drop down side rails
- 4" high-density litter pad included
- Four-wheel central lock and brake

WARRANTY:

Two-year parts & labor warranty on the frame & the top. One-year warranty on all electrical components.

MRI TRANSPORT: 250 - 350 LB. WEIGHT CAPACITY



MRI Transport Wheelchair

- **Weight Limit: 250 – 350 pounds**
- Seat Widths:
18" for 250 lbs., 20" for 350 lbs.
- Seat Depth: 18"
- Seat to Floor Height: 19"
- Back Style: Permanent
- Swing back arms and swing away foot rests.
- Non-magnetic, stainless steel construction

WARRANTY:

Limited lifetime warranty on side frames & cross braces. One-year warranty on all other components.



MRI Transport Stretcher

- **Weight Limit: 350 pounds**
- Patient Surface:
Width: 23"
Length: 76"
- Stretcher includes side rails, manual fowler back rest and mattress.
- Non-magnetic, stainless steel construction

WARRANTY:

Two-year parts & labor warranty on the frame & the top. One-year warranty on all electrical components.

PRONE STRETCHER



Regency Plus™ Prone Stretcher

- **Weight Limit: 250 pounds**
- Patient surface 22" wide x 72" long
- Height adjusts (manually) 24" to 30"
- Chin wedge and pad included
- User-accessible wheel locks
- Full non-marring bumper

**TWO-YEAR
WARRANTY**



Sales and Service of Cardiology and Surgical Equipment and Supplies

BARIATRIC PRODUCTS CATALOG

